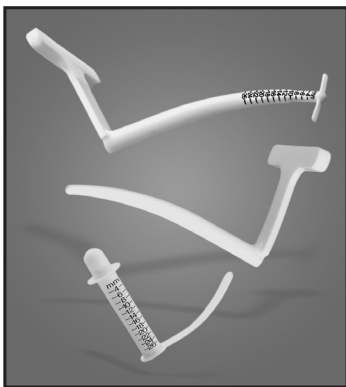
# INHEALTH<sup>®</sup>

TECHNOLOGIES

## Blom-Singer<sup>®</sup> Dilation/Sizing System

USER Instructions For Use

**Blom-Singer<sup>®</sup>**

voice restoration systems

**R1**

37685-01H

37685-01H Effective October 2017 / Gültig ab Oktober 2017 / Efectivo a partir de octubre de 2017 / Date d'entrée en vigueur : octobre 2017 / Valido da ottobre 2017 / Met ingang van oktober 2017 / Vigente em outubro de 2017 / V platnosti od října 2017 / Wchodzi w życie w październiku 2017 r.

**Blom-Singer** and **InHealth Technologies** are registered trademarks of **Freudenberg Medical, LLC**

Patent(s): [www.inhealthpatents.com](http://www.inhealthpatents.com)

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## **BLOM-SINGER® DILATION/SIZING SYSTEM**

The Blom-Singer Dilation/Sizing System is designed to facilitate dilating and measuring of the tracheoesophageal puncture tract (TEP) in preparation for insertion of a Blom-Singer voice prosthesis. Included in the system is a Puncture Dilator that also prevents closure of the tracheoesophageal puncture and aspiration of fluids when the voice prosthesis is removed.

### **INDICATIONS**

The Blom-Singer Dilation/Sizing System is indicated for dilation, measuring, and maintaining an open passage of the tracheoesophageal puncture for tracheoesophageal voice restoration following total laryngectomy. The Blom-Singer Sizing System is a measurement device used by physicians and qualified speech pathologists to select the appropriate voice prosthesis length.

### **CONTRAINDICATIONS**

The components of the Blom-Singer Dilation/Sizing System are medical devices and should only be used by individuals with experience and training in the use and care of tracheoesophageal puncture prosthetic devices.

These devices should not be placed in a small tracheostomal airway if its presence obstructs breathing. These devices are not valved voice prostheses and do not function like one.

Placement of the 18 French Puncture Dilator or Dilator-Sizer is contraindicated in a tracheoesophageal puncture that has dilated substantially beyond the diameter of a conventional 16 French Blom-Singer voice prosthesis. Likewise, the 22 French Puncture Dilator or Dilator-Sizer should not be used in a puncture that has dilated substantially beyond the diameter of a 20 French Blom-Singer voice prosthesis.

### **HOW SUPPLIED**

The components of the Dilation/Sizing System are supplemental devices to the Blom-Singer Voice Restoration System. The system

consists of three separate products, as shown in the chart below, that are available nonsterile. These products may be sterilized, see sterilization and reuse instructions.

Dilation/ Sizing System Components	Clinician Use	Patient Use	Size	Use With
Puncture Dilator	•	•	18 Fr.	16 Fr. Voice Prosthesis
			22 Fr.	20 Fr. Voice Prosthesis
Voice Prosthesis Sizer	•		16 Fr.	
			20 Fr.	
Dilator-Sizer	•		18 Fr.	16 Fr. Voice Prosthesis
			22 Fr.	20 Fr. Voice Prosthesis

## PRODUCT DESCRIPTION

*Please refer to the diagrams located at the back of this instruction manual.*

### Puncture Dilator

The Blom-Singer Puncture Dilator (diagram 1) is a tapered, solid silicone stent that is inserted into the tracheoesophageal (TE) puncture upon removal of the voice prosthesis. Its purpose is to prevent the puncture from closing; prevent leakage of esophageal contents into the trachea during swallowing; and dilate the puncture to ease voice prosthesis insertion. This device can also be placed for longer intervals in emergency situations when the voice prosthesis is unavailable.

This device is available for use by the patient and/or clinician. The user should always have a Puncture Dilator, of the appropriate diameter, readily available.

### Voice Prosthesis Sizer

The Blom-Singer Voice Prosthesis Sizer (diagram 2) is used to measure the length of the tracheoesophageal puncture tract between the trachea and the esophagus to determine the

---

appropriate voice prosthesis length. Incremental millimeter markings and numbers on the device correspond to the lengths of Blom-Singer Voice Prostheses. The product is packaged with three sizer inserters and replacement inserters are also available separately. This device is only available for use by a qualified, trained medical professional.

### **Dilator-Sizer**

The Blom-Singer Dilator-Sizer (diagram 3) is a combination Puncture Dilator and Voice Prosthesis Sizer. The Dilator-Sizer is used to dilate the puncture to ease voice prosthesis insertion; and as a Voice Prosthesis Sizer, it is used to measure the length of the tracheoesophageal puncture tract between the trachea and the esophagus to determine the appropriate voice prosthesis length. Incremental millimeter markings and numbers on the device (diagram 4) correspond to the lengths of Blom-Singer Voice Prostheses. This device is only available for use by a qualified, trained medical professional. Patent pending.

### **INSTRUCTIONS FOR USE**

*The following procedure is provided by Eric D. Blom, Ph.D. Please refer to the diagrams located at the back of this instruction manual.*

The Blom-Singer Dilation/Sizing device and user's hands should always be thoroughly clean to help avoid introducing contaminants into the tracheostoma, puncture, or esophagus. Do not use solvents or petroleum-based products for cleaning these devices. These materials may damage silicone (see cleaning and sterilization instructions).

Removal and insertion of devices in the puncture should only be done with the user positioned in front of a mirror with a bright light focused directly into the stoma. Care should be exercised to avoid inhaling prosthetic devices into the airway. In the unlikely event that this should occur, the patient should attempt to cough the device out of the windpipe and immediately seek medical attention if unsuccessful.

### **Dilation of the TE Puncture (User/Clinician)**

Dilate the TE puncture with either the Puncture Dilator (User) or the Dilator-Sizer (Clinician) of the appropriate diameter (see

sizing chart). Position the tapered tip of the dilator in the puncture and gently insert the device completely to the neck strap (diagrams 1a, 3a, 5, 6). Avoid touching the portion of the dilator that will enter the puncture. Secure the dilator near the edge of the stoma with adhesive tape across the neck strap to prevent the dilator from slipping out. Note: if the puncture angles straight back or slightly downward, insert the dilator such that its curvature is also directed downward. If the puncture angles in an upward plane, insert the dilator such that its curvature is also directed in an upward direction. Remove the dilator if gagging or discomfort is experienced; users should consult with their physician or speech pathologist.

The dilator should be left in place for 5 minutes, or longer if necessary, in order to sufficiently dilate the TE puncture and allow for easy insertion of the voice prosthesis (Clinician: or Voice Prosthesis Sizer). Remove the tape from the neck strap, grasp the dilator securely, and carefully withdraw it from the puncture. Immediately insert a voice prosthesis (Clinician: or Voice Prosthesis Sizer).

### **Measurement of the TE Puncture Tract (Clinician)**

**Note:** Always measure the TE puncture tract to confirm correct voice prosthesis length *before* inserting a voice prosthesis.

#### **Dilator-Sizer**

The Dilator-Sizer, of the appropriate diameter (see sizing chart), is inserted gently by the Clinician completely into the TE puncture (diagram 6). Note: if the puncture angles straight back or slightly downward, insert the dilator such that its curvature is also directed downward. If the puncture angles in an upward plane, insert the dilator such that its curvature is also directed in an upward direction. Remove the dilator if the patient experiences gagging or discomfort.

Carefully withdraw the Dilator-Sizer until very gentle resistance is detected indicating that the esophageal wings (diagram 4) are seated against the anterior wall of the esophagus (diagram 10). Note: Pulling too firmly on the Dilator-Sizer may squeeze or compress the TE tract resulting in an erroneous measurement that indicates it is shorter than it actually is.

---

Note the marker that is closest to the entrance of the TE puncture. Read the size number on the Dilator-Sizer and select a Blom-Singer voice prosthesis of corresponding size. If the measurement is between two markers, select the longer one.

Fully insert the Dilator-Sizer and fasten the neck strap with adhesive tape on to the patient's neck. Leave the Dilator-Sizer in the puncture to dilate 5 minutes, or longer if necessary, in order to allow for easy insertion of the selected voice prosthesis. Prior to insertion of the Blom-Singer voice prosthesis, remove the tape from the neck strap, grasp the dilator securely, and carefully withdraw it from the puncture. Immediately insert the voice prosthesis.

### **Voice Prosthesis Sizer**

The Voice Prosthesis Sizer (diagram 2) is inserted by the Clinician into the TE puncture until it touches the posterior esophageal wall, directed down the esophagus, and gradually withdrawn up and out of the puncture until very gentle resistance of the esophageal collar (diagram 2a) against the anterior esophageal wall is detected. The sizing marker visualized nearest to but outside the puncture site at the posterior tracheal wall determines prosthesis length. Note: Pulling too firmly on the Voice Prosthesis Sizer may squeeze or compress the TE tract resulting in an erroneous measurement that indicates it is shorter than it actually is.

1. Place the tip of the inserter into the open end of the Voice Prosthesis Sizer (diagram 7). Hold the Voice Prosthesis Sizer by the neck strap (diagram 2b) to avoid touching the portion that inserts into the TE puncture.
2. Attach the neck strap on the Voice Prosthesis Sizer securely over the safety peg (diagram 7a) on the inserter (diagram 8). This should prevent inadvertent dislodgment of the device during insertion into the TE puncture.
3. Hold the end of the inserter (diagram 9) and insert the Voice Prosthesis Sizer completely into the TE puncture. Next, carefully withdraw the Voice Prosthesis Sizer until very gentle resistance is detected indicating that the esophageal collar is seated against the anterior wall of the esophagus (diagram 11).

4. Note the marker that is closest to the entrance of the TE puncture (diagram 12). Read the size number on the Voice Prosthesis Sizer and select a Blom-Singer Voice Prosthesis of corresponding size. If the measurement is between two markers, select the longer one.

### **DEVICE CLEANING: Puncture Dilator**

The Puncture Dilator may be cleaned and reused by a single patient as follows: Clean the Puncture Dilator thoroughly with a mild detergent and rinse under a strong flush of tap water. Dry the dilator with sterile gauze. Once a dilator has been cleaned and dried completely, it should be stored in a clean, reclosable plastic bag until its next use. Always handle with clean hands and avoid touching portions of the dilator that will enter the puncture.

**Precaution:** Never clean the device while the dilator is in position in the neck. Always clean the dilator after it has been removed.

## **INSTITUTION STERILIZATION AND REUSE INSTRUCTIONS**

The devices of the Blom-Singer Dilation/Sizing System may be cleaned and steam sterilized for reuse by multiple patients as follows: Clean thoroughly with a mild detergent and rinse with water. Sterilize unwrapped in a standard autoclave at 132—135°C for 3 minutes. Do not sterilize the device in the original packaging. The device may be reused to a limit of 100 times. Inspect the device routinely for structural damage. If there are tears, cracks, or structural damage, do not continue to use. Do not sterilize the sizer inserter.

## **WARNINGS AND PRECAUTIONS**

The components of the Blom-Singer Dilation/Sizing System are medically prescribed devices. Initial sizing and instructions in their use must be provided by a qualified physician or speech pathologist trained in the use of these particular devices.

The Blom-Singer Voice Prosthesis Sizer and Dilator-Sizer are medical devices and should be used only by a physician or licensed speech pathologist trained in their use. The Voice Prosthesis Sizer is not a valved voice prosthesis and cannot function as such.

If there are any tears, cracks, or structural damage, do not continue to use this device.

---

Never clean the Puncture Dilator device while the dilator is in position in the neck. Always clean the dilator after it has been removed.

**Caution:** Federal (USA) law restricts this device to sale by or on the order of a physician or licensed speech pathologist.

## **COMPLICATIONS**

Although rare, the following complications have been identified to occur with silicone prosthetic devices of the Blom-Singer type. They include: stoma contamination or infection, which may require removal of the dilator and/or use of appropriate antibiotics; accidental inhalation of a prosthetic device into the airway, which may require removal by a physician; inhalation of liquids through a puncture into the airway, which may cause the patient to cough; inflammatory reaction around the puncture site and formation of granulation tissue; puncture dilation resulting in leakage of fluids around the device; dislodgment of the device and subsequent closure of the tracheoesophageal puncture; intractable leakage around the device, requiring surgical revision or closure of the puncture; dysphagia, or difficulty swallowing.

## **ORDERING INFORMATION**

### **USA**

Blom-Singer products may be ordered directly from InHealth Technologies. TELEPHONE: Toll-Free (800)477-5969 or (805)684-9337, Monday–Friday, 9:30am–7:00pm, Eastern Standard Time. FAX: Toll-Free (888)371-1530 or (805)684-8594. EMAIL: [order@inhealth.com](mailto:order@inhealth.com) ORDER ON-LINE: [www.inhealth.com](http://www.inhealth.com) POST: InHealth Technologies, 1110 Mark Avenue, Carpinteria, CA 93013-2918, USA, Attention: Customer Service

### **International**

Please contact our customer service department for a distributor referral.

### **Consumer Affairs**

If you have any questions or dissatisfaction with a product please contact our customer service department via telephone, fax, post, or Email: [productcomplaints@inhealth.com](mailto:productcomplaints@inhealth.com)

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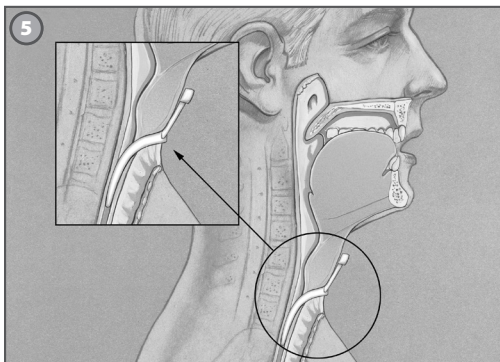
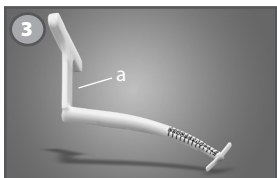
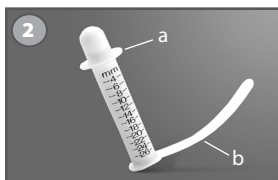
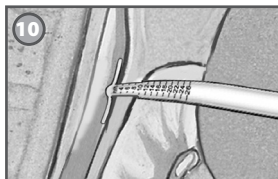
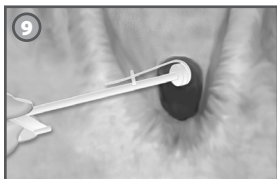
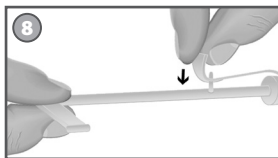
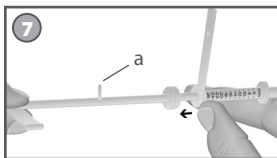
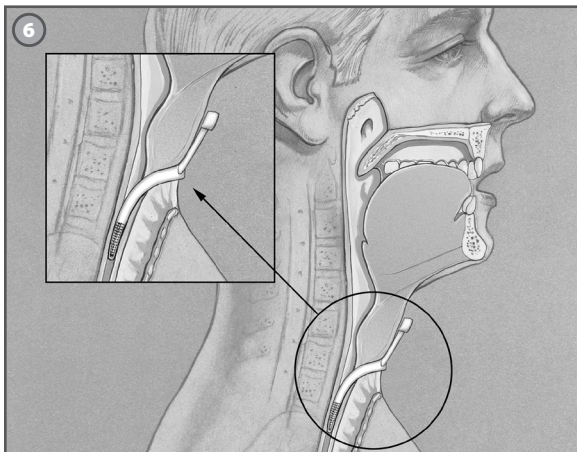
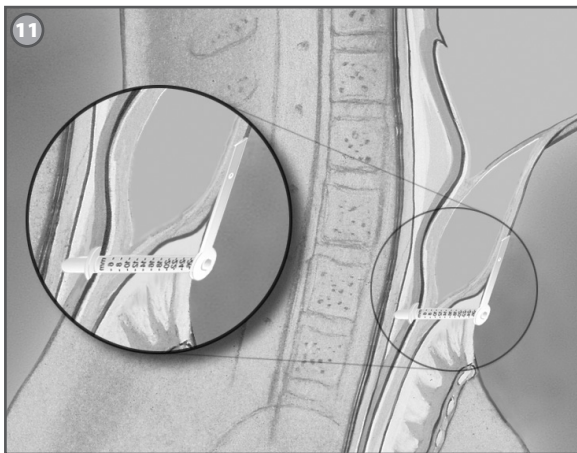
Additional references available upon request.

Zusätzliche Hinweise sind auf Rückfrage verfügbar. / Las referencias adicionales se encuentran disponibles a pedido. / Des références supplémentaires sont disponibles sur demande. / Riferimenti aggiuntivi disponibili su richiesta. / Op verzoek zijn meer referenties verkrijgbaar. / Referências adicionais disponíveis mediante solicitação. / Další odkazy na literaturu jsou k dispozici na požádání. / Dodatkowe informacje dostępne na żądanie.

1. Blom, E.D.: "Some Comments on the Escalation of Tracheoesophageal Voice Prosthesis Dimensions," **Arch Otolaryngol**, 129: 500-502, 2003.
2. Eerenstein, S.E., Grolman, W., Schouwenburg, P.F.: "Downsizing of Voice Prosthesis Diameter in Patients with Laryngectomy," **Arch Otolaryngol**, 128: 838-841, 2003.
3. Hamaker, R.C., Blom, E.D.: "Botulinum Neurotoxin for Pharyngeal Constrictor Muscle Spasm in Tracheoesophageal Voice Restoration," **Laryngoscope**, 113:1479-1482, 2003.
4. Evans, P.H.R., Blom, E.D.: "Functional Restoration of Speech," **Principles and Practice of Head and Neck Oncology**, Evans, P.H.R., Montgomery, P.W., Gullane, P.J. (Eds.) Taylor and Francis Group, London, 2003.
5. Blom, E.D., Singer M.I., and Hamaker, R.C., **Tracheoesophageal Voice Restoration Following Total Laryngectomy**. San Diego: Singular Publishing Group, Inc.1998.
6. Blom, E.D., Hamaker, R.C.: "Tracheoesophageal Voice Restoration Following Total Laryngectomy," **Cancer of the Head and Neck**, E.N. and Suen, J. (Eds). W.B. Saunders Publishers. 839-852, 1996.
7. Blom, E.D., "Tracheoesophageal Speech," **Seminars in Speech and Language**. New York: Thieme Medical Publishers. 16:3, 191-204, 1995.
8. Pauloski, B.R., Blom, E.D., Logemann, J.A., Hamaker, R.C. "Functional Outcome After Surgery for Prevention of Pharyngospasms in Tracheoesophageal Speakers. Part II: Swallow Characteristics," **Laryngoscope**, 105:10: 1104-1110, 1995.
9. Blom, E.D., Pauloski, B.R., Hamaker, R.C. "Functional Outcome After Surgery for Prevention of Pharyngospasms in Tracheoesophageal Speakers. Part I: Speech Characteristics," **Laryngoscope**, 105:10: 1093-1103, 1995.

- 
10. Blom, E.D., Hamaker, R.C., Freeman, S.B.: "Postlaryngectomy Voice Restoration," Lucente, F.E. (Ed.): **Highlights of the Instructional Courses**, Vol. 7. St. Louis: Mosby-Year Book. pp 3-10, 1994.
  11. Blom, E.D., Singer M.I., and Hamaker, R.C. "A Prospective Study of Tracheoesophageal Speech," **Arch Otolaryngol**, 112: 440-447, 1986.
  12. Blom, E.D., Singer, M.I. "Disinfection of Silicone Voice Prostheses," **Arch Otolaryngol**, 112: 1986.
  13. Blom, E.D., Singer M.I., and Hamaker, R.C. "An Improved Esophageal Insufflation Test," **Arch Otolaryngol**, 111: 211-212, 1985.
  14. Hamaker, R.C., Singer M.I., Blom E.D. and Daniels, H.A. "Primary Voice Restoration at Laryngectomy," **Arch Otolaryngol**, 111: 182-186, 1985.
  15. Singer, M.I. and Blom, E.D. "A Selective Myotomy for Voice Restoration After Total Laryngectomy," **Arch Otolaryngol**, 107: 670-673, 1981.
  16. Singer, M.I. and Blom, E.D. "An Endoscopic Technique for Restoration of Voice After Laryngectomy," **Ann Otol Rhinol Laryngol**, 89:6: 529-533, 1980.

# INHEALTH®

TECHNOLOGIES

---

## UNITED STATES

1110 Mark Avenue  
Carpinteria, CA 93013-2918 USA  
Toll-Free: 800.477.5969  
Toll-Free Fax: 888.371.1530

## INTERNATIONAL

Tel: +1.805.684.9337  
Fax: +1.805.684.8594  
E-mail: [order@inhealth.com](mailto:order@inhealth.com)

## ORDER ONLINE

[www.inhealth.com](http://www.inhealth.com)

**Freudenberg Medical, LLC**  
1110 Mark Avenue  
Carpinteria, CA 93013-2918 USA  
[www.inhealth.com](http://www.inhealth.com)

CE 0344

**EMERGO EUROPE**  
Prinsessegracht 20  
2514 AP The Hague  
The Netherlands