# VOICETEC® SPRACHVERSTÄRKER

---

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# VOICETEC® VOICE AMPLIFIER

EN

The Voicetec® voice amplifier (REF 77350) is an aid for patients with limited speaking volume. It enables very good amplification of the quiet oesophageal voice, in laryngectomised patients, for example. It is especially well suited whenever a louder conversation volume is desired, in order to also be understood even with high levels of background noise. With the aid of the Voicetec® voice amplifier, this is also possible without strenuous attempts to speak loudly.

The handy, wireless and radio-controlled transmitter unit that can be attached to clothing, for instance, enables a great deal of mobility. This extends the patient's comfort and field of application.

Please read carefully before using the device for the first time. Keep in a safe place for future use.

## 1. Safety information

### 1.1 Purpose

The operational safety of the Voicetec® voice amplifier is only guaranteed if it is used as intended.

#### Intended use

The intended use is given only if

- the Voicetec® voice amplifier is used exclusively in dry rooms
- the Voicetec® voice amplifier is used in an undamaged state
- the Voicetec® voice amplifier is used in accordance with these Operating Instructions

#### Non-intended use and therefore prohibited includes:

- operation of the Voicetec® voice amplifier in a damaged state
- interference or modification of the Voicetec® voice amplifier without the manufacturer's authorization
- use of the Voicetec® voice amplifier in the shower/bath
- disregard of the Operating Instructions

**Non-intended use can lead to serious injury or damage to property!**

### 1.2 Special safety information

**Only operate the device with the original power supply connector. In the event of a fault, you have to pull the power supply connector out of the mains socket to ensure complete disconnection from the mains!**

Interrupt operation of the Voicetec® voice amplifier immediately if a solid object or fluid enters the device.

In this event, immediately pull the power supply connector out of the mains socket and have the device checked by our customer service.

Pull the power supply connector out of the mains socket if the device is unused for long periods of time. Please note that if you switch off the device and the power supply connector is connected, the device is not completely discontinued from the mains.

Do not place the device near heat sources, such as radiators, heating pipes, amplifiers, etc. and do not expose it to direct sunlight, intense dust and moisture, rain, vibrations or impacts.

### 1.3 Inspections & tests

Test the following points prior to use:

- Does the operating voltage specified on the power supply connector match the mains voltage at the operating site?

### 1.4 Repair and spare parts

**Caution!**

**There is a risk of injury due to inadequate or unsuitable accessories. Only use the manufacturer's original parts!**

Unauthorised interference with the device, modification or changes to the Voicetec® voice amplifier invalidates all warranty claims and the manufacturer's liability. The same applies for the use of non-original parts.

- Repairs and maintenance must only be undertaken by the manufacturer or by parties authorised by the manufacturer.

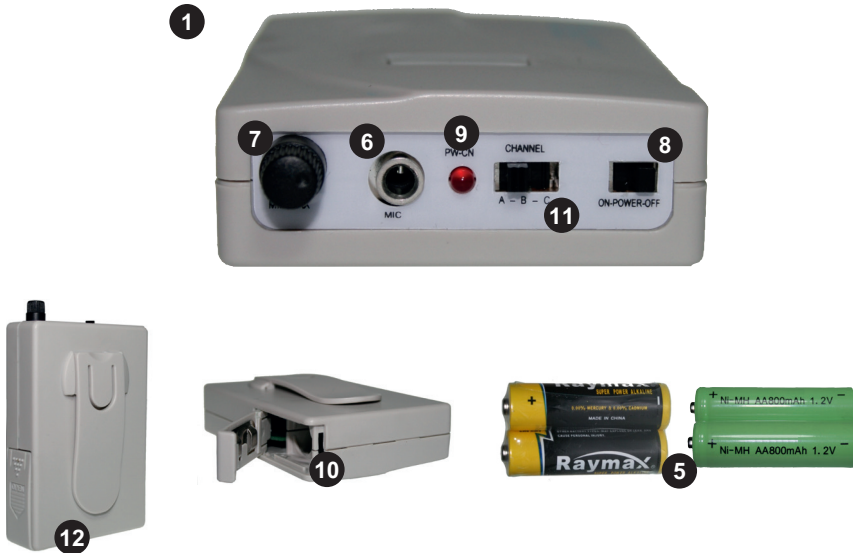
### 1.5 Product observation

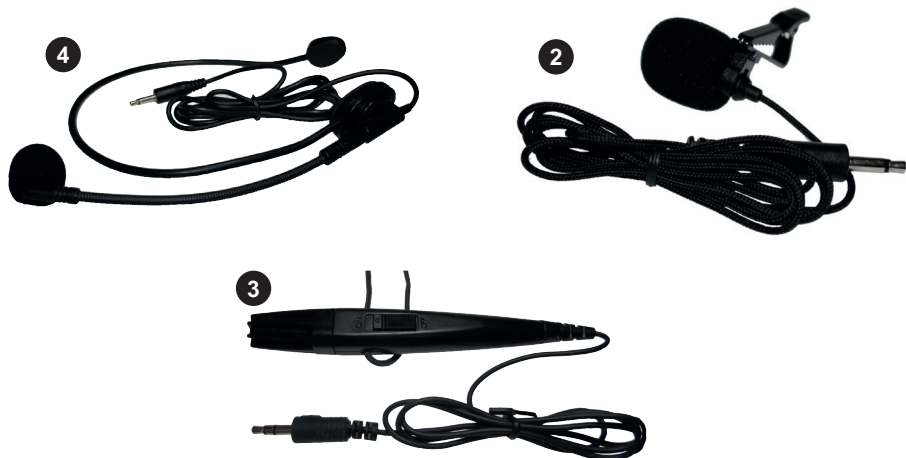
We would like to offer you safe, state-of-the-art products in the future too. Share your suggestions with us and inform us about problems that arise with the Voicetec® voice amplifier.

## 2 Description of the voice amplifier


The aim of this chapter is to familiarise the user with the design of the Voicetec® voice amplifier and the display and operation elements.

### 2.1 Device overview / Transmitter unit scope of delivery

## 2.2 Device overview / receiver unit scope of delivery

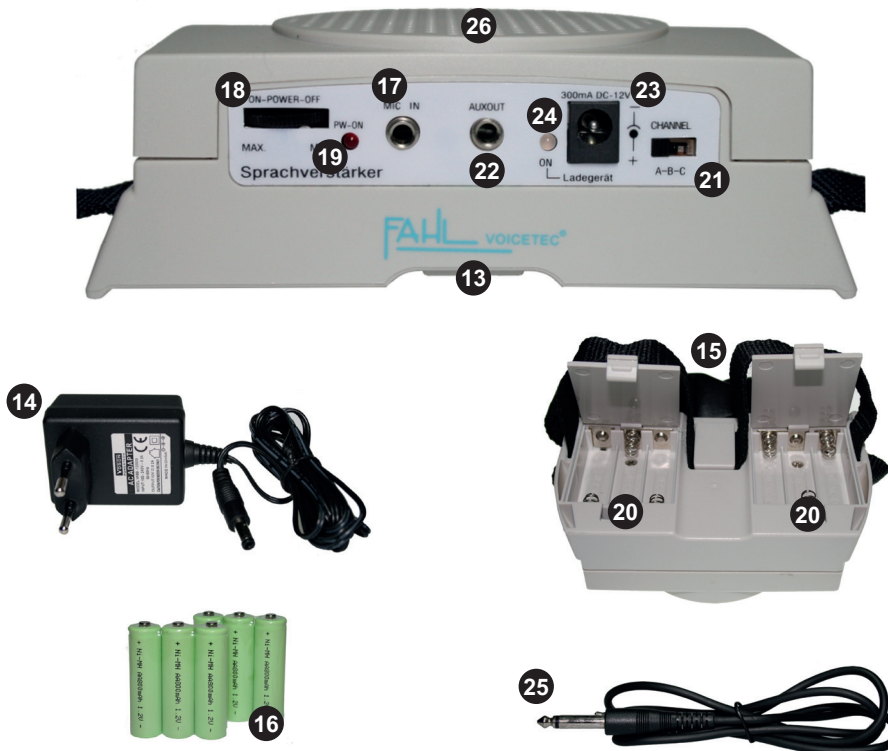Replacement batteries are included in the scope of delivery for the receiver unit.

## 2.3 Transmitter unit functional description

Pos.	Name	Function
1	Transmitter unit	Sends voice as a radio signal to the receiver unit
2	Clip-on microphone	Microphone to pin onto clothing using an attachment clip
3	Hand microphone	Miniature microphone to hold in the hand or hang around the neck
4	Microphone headband	Microphone worn on the head
5	1.5 V batteries or rechargeable batteries	Batteries as the power supply for the transmitter unit
6	Microphone input socket	Input socket to connect the microphone
7	Volume control	Volume setting knob
8	On / Off switch	Switches the transmitter unit on and off
9	Control lamp	Signals whether the transmitter unit is switched on
10	Battery compartment	Battery compartment to connect the batteries
11	Channel switch	Switch to select the radio channel
12	Retaining clip	Retaining clip to attach the transmitter unit to the body

## 2.4 Receiver unit functional description

Pos.	Name	Function
13	Receiver unit / amplifier	Receives the signals from the transmitter unit and amplifies speech
14	Power supply connector	Charges the rechargeable batteries of the receiver unit
15	Carrying strap	Carrying strap to attach the receiver unit to the body
16	Rechargeable batteries	6 Batteries to supply the receiver unit
17	Microphone input socket	Input socket to connect the microphone
18	Volume control - On / Off switch	Knob to switch the receiver unit on / off and set the volume
19	Control lamp	Signalises whether the transmitter unit is switched on
20	Battery compartments	Battery compartment to connect the rechargeable batteries
21	Channel switch	Switch to select the radio channel

Pos.	Name	Function
22	Audio output socket	Socket to connect an external amplifier or audio devices
23	Charging socket	Socket to connect the power supply connector
24	Charge status lamp	Signalises the charge status
25	Audio connecting cable	Connecting cable to connect an external amplifier or audio devices
26	Loudspeaker	Amplifies the voice

## 2.5 Technical data

Description	Values
<b>Transmitter unit</b>	
Power supply	2x 1.5 V type AA or 2x 1.2 V 1000 mAh Ni-MH type AA
Power consumption	120 mA
Transmission frequency	2.4 GHz
Housing dimensions	90 x 60 x 24 mm
Weight	70 g (without batteries)
Operating time	Around 8-20 hours (dependent on battery type)
<b>Receiver unit</b>	
Power supply connector	AC 110-240 V 300 mA / 50-60 Hz
Power supply	6x 1.2V 1000mAh Ni-MH type AA
Receiving frequency	2.4 GHz
Range	approx. 10 m
Output power	5 W max. to 4 Ω
Housing dimensions	140 x 80 x 45 mm
Weight	270 g (without rechargeable batteries)
Operating time	Around 6-10 hours (depending on the volume set)
Charging time	Around 5-7 hours
<b>General</b>	
Operating temperature	-10°C to +50°C
Device class	Type B
Protection class	II
Marking	CE for medical devices

Subject to alterations.

### 3 Operation of the voice amplifier

This chapter informs you about putting the Voicetec® voice amplifier into operation safely for the first time.

#### 3.1 General prerequisites

The Voicetec® voice amplifier is ready for operation after removing from the packaging and observing the following provisions.

- Carry out a visual inspection for external damage. The device must be free of damage.
- If there is any damage, do not put the Voicetec® voice amplifier into operation.

#### 3.2 First use

Check whether all parts are included in the scope of delivery in accordance with the device overview (see chapter 2.2 / 2.3 of the Operating Instructions).

**Please check that the transmitter unit ❶ and the receiver unit ❷ are working on the same radio channel. Set the channel switch (❸ or ❹) to the same value (A or B or C) for this purpose. Transmission is only possible with the same radio channel. If the connection on one channel is faulty, change both devices to another value. If the transmitter and receiver units are not set to the same radio channel, the control lamp ❺ on the transmitter unit flashes green.**

#### 3.3 Preparing the transmitter unit for operation

- Inserting and changing batteries

You can optionally use two batteries (1.5 V type AA) or two rechargeable batteries (1.2 V Ni-MH type AA) (included in the scope of delivery) for operation. The rechargeable batteries can be charged in the battery compartment ❷ of the receiver unit (see description in chap. 3.4) or with any commercially available charger. Press on the release button of the battery compartment ❸ and fold open the cover. Insert the batteries ❹. Ensure that the polarity is correct when inserting. You can identify the polarity from the sticker inside the battery compartment. Close the battery compartment. The cover locks in place audibly.

- Connect the microphone

Connect one of the three microphones ❶, ❷, ❸ supplied. Connect the socket plug of the microphone with the microphone input socket ❹ of the transmitter unit.

- Switch on the transmitter unit

Switch on the transmitter unit with the On /Off switch ❶. The control lamp ❷ of the transmitter unit must now light green.

- Setting the microphone

Depending on which microphone you use, configure it such that it is as close as possible to the tracheostoma. You can set the volume of the transmitter unit with the volume control ❶. Turning to the left reduces the volume, to the right increases the volume.

#### 3.4 Preparing the receiver unit for operation

- Inserting and changing rechargeable batteries

Only rechargeable batteries (1.2 V Ni-MH type AA) may be used for operation.

**Do not use normal 1.5 V type AA batteries. The receiver unit may be destroyed on connecting the power supply connector ❶.**

Loosen the carrying strap 15 and open up the two covers of the battery compartments 20. Insert the six rechargeable batteries 16 in the two battery compartments. Ensure that the polarity is correct when inserting. Close the battery compartments. The covers lock in place audibly.

- Charging rechargeable batteries

Connect the power supply connector 14 with the charging socket 23, then connect the power supply connector with a suitable mains socket. You can connect the power supply connector to a 100-240 V / 50-60 Hz mains socket.

- Signalising the charging process

Next to the charging socket there is a control lamp 24, with which the charging status is displayed. During the charging process, the charging control lamp lights red. Once the charging process is concluded, the charging control lamp lights green.

### Continued

- Rechargeable battery charging time

The rechargeable batteries 16 are in a charged state when delivered, but it is recommended to charge them fully prior to using for the first time.

Standard charge: around 5 hours.

- Switch on the receiver unit

Switch on the receiver unit with the volume control On/Off switch 18. The control lamp 19 must now light green.

- Setting volume

You can set the volume on the receiver unit with the volume control On/Off switch. Turning to the left reduces the volume, to the right increases the volume.

**Make sure that the overall volume is also influenced by the setting of the transmitter unit volume control 7.**

## 3.5 Operating without a transmitter unit

The receiver unit can also be used without a transmitter unit. For this purpose, connect any one of the three microphones 2, 3, 4 supplied directly with the receiver unit 13. Connect the microphone socket plug with the microphone input socket 17 of the receiver unit.

## 3.6 Connection to an external amplifier

You can also connect the Voicetec® voice amplifier to an external amplifier or an audio device. Connect the audio connecting cable 25 with the audio output socket 22 of the receiver unit and the external amplifier or the audio device. If the audio connecting cable is plugged into the audio output socket of the receiver unit, the loudspeaker 26 of the receiver unit is switched off. Speech reproduction is now only via the external amplifier or the audio device.

## 3.7 Operating time

The operating time of the transmitter unit is approx. 8-20 hours depending on the battery type. If the capacity of the batteries is no longer sufficient, the colour of the control lamp 9 changes to red. You have to replace or recharge the batteries.

The operating time of the fully charged receiver unit (set to medium volume) is approx. 6-10 hours. If the capacity of the rechargeable batteries is no longer sufficient, the control lamp (19) changes to red. You have to recharge the batteries.

### 3.8 Information on the rechargeable batteries

The rechargeable batteries maintain their capacity if they are stored at 21°C. If the rechargeable batteries are not used, they have to be recharged after 3 months, as otherwise they can be damaged due to deep discharge.

In order to maintain the maximum capacity of the rechargeable batteries and to exploit the maximum number of charging cycles, it is recommended to use the rechargeable batteries until the control lamp  lights red.

The power supply connector  and the rechargeable batteries can heat up during the charging process. This is normal and does not constitute a fault.

**The rechargeable batteries are consumables. The service life depends on the number of charges, proper storage and observance of the recommended usage. The warranty period for the rechargeable batteries is 6 months.**

## 4 Cleaning and care

Clean the device with a damp, but not wet cloth. Always pull the power supply connector out of the mains socket beforehand! Do not use any sharp or aggressive detergents or any that contain alcohol or solvents, as they could damage the plastic parts.

The device is suitable for reconditioning and reuse. The housing parts have to be disinfected with domestic surface disinfectants for this purpose. Take care that no disinfectant solution enters the device sockets or the loudspeaker .

**The microphones have to be replaced before every change of patient. No reconditioning is possible here.**

## 5 Faults, troubleshooting

EN

### 5.1 Faults that you, as a user, are allowed to rectify

Fault	Possible cause	Actions of the user/patient
No sound.	Transmitter and receiver units have different frequencies.	Set the channel switch to the same position.
	Transmitter and receiver units are switched off.	Switch on the transmitter and receiver units.
	Volume control is on zero.	Turn the volume control to the right.
	Microphone is not connected.	Connect the microphone.
	Batteries inserted incorrectly.	Insert batteries correctly. Observe the "+" and "-" signs!
	Batteries are empty.	Replace or charge the batteries.
	Transmitter unit is too distant.	Place the transmitter unit closer (within the range of 15 m).
Distortions, noise, crackling, unwanted signals.	Störungen durch andere Sprachverstärker, Fernsehen, Radio, Funkgeräte oder schadhafte Elektrogeräte.	Set the channel switch to another channel.
		Switch off interfering or damaged devices Have domestic electrical installation checked by an external specialist company.

If these measures prove unsuccessful, please send the device to Andreas Fahl Medizintechnik-Vertrieb GmbH for examination / repair.

Never undertake your own repairs. The device must only be opened by the manufacturer or by its authorized companies / persons.

## 6 Spare parts for Voicetec®

Pos. *	Part name	Item no.
2	Clip-on microphone	77350-01
3	Hand microphone	77350-02
4	Microphone headband	77350-03
14	Power supply connector	77350-04
15	Carrying strap	77350-05
16	Rechargeable batteries for receiver unit	77350-06
25	Audio connecting cable	77350-07

See part number according to the device overview chapter 2.1 and 2.2.

## 7 Disposal / Environment

The power supply connector draws a low current even if the device is switched off. Therefore, to save energy, pull the power supply connector out of the mains socket if you are not using the device for a prolonged period.

The Voicetec® voice amplifier is designed for use by the end user / patient.

The voice amplifier is subject to the German Electrical Equipment Act (ElektroG), so special duties apply with regard to return and disposal. Please inform us when the device is no longer needed. We would be pleased to advise you on the further procedure.

**The following instructions must be observed for disposing of the voice amplifier:**

**Note!**

**Observe the relevant statutory requirements in accordance with ElektroG, the German Medical Products Act (MPG), etc. Information is available to you on the internet on the website from the joint body, the German Electrical Used Equipment Register (EAR), commissioned by the competent authority (German Federal Environment Ministry): [www.stiftung-ear.de](http://www.stiftung-ear.de)**

- The Voicetec® voice amplifier must be disposed of as electrical waste and must not go into normal household waste.
- The public waste disposal facilities (municipal waste management companies) have set up special collection points for used electrical devices. Here consumers can hand in / deliver used devices free of charge. You can enquire to your local waste disposal facility (waste management company) about a collection point close to where you live.

## 8 Manufacturer / Conformity

The Voicetec® voice amplifier is a class I medical device pursuant to Rule 1 of the European Medical Device Directive 93/42/EEC.

The device conforms with the German Medical Products Act (MPG) and the European Medical Device Directive.

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**<http://www.fahl.de>**

Device serial number - see rating plate.

## Device certificate

With the Voicetec® voice amplifier you have purchased a high quality device. Should the device nevertheless fail to work perfectly within 24 months of the purchase date, we will repair or replace it free of charge.

The warranty period for the rechargeable batteries is 6 months.

Damage or faults attributable to improper use or technical interference by parties who have not been authorized by us, lead to termination of the warranty claim.

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SN device (see rating plate) \_\_\_\_\_

Date of delivery \_\_\_\_\_

In case of faults, return the product to us with a copy of the completed device certificate to:

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**August-Horch-Straße 4a  
51149 Cologne**

Modifications and repairs may only be carried out by the manufacturer or by a body authorised by the manufacturer.

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