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The instructions for service to help the physician, nursing staff and patient/user in order to ensure technically correct handling.

Please read the instructions for use carefully before using the product for the first time!

**II. INTENDED USE**  
LARYNGOFIX®/TRACHEOPIX®/TRACHEOSTOMA-FIX® filter the breathing air, thereby reducing intrusion of particles into the patient's airways.

**III. WARNINGS**  
LARYNGOFIX®/TRACHEOPIX®/TRACHEOSTOMA-FIX®/LARYTAPE® are single-patient products and are intended for single use only.

It must not be cleaned or disinfected, as use of cleaning agent can render the filter medium unusable.

Sterilisation and reuse can impair the function and is therefore not permissible either.

Should there be any irritation (e.g. redness, itchiness), discontinue the product immediately and consult a physician. This may be due to special skin sensitivity or allergy-related incompatibility.

**IV. INSTRUCTIONS**

Clean the skin around the tracheostoma in front of a mirror using warm water and pH-neutral soap or OPTIFAHLL® stoma cleaning cloths. Dry the skin. Before cleaning, remove residues of the skin if necessary using OPTICLEAR®.

After removing the protective foil ☀ place the tracheostoma protection centre over the tracheostoma ☀ and cover the tracheostoma this way ☀.

Remove the tracheostoma protection ☀ by carefully removing the adhesive strip from the skin.

**LARYNGOFIX® TOUCH/LARYTAPE® TOUCH:**

For speaking, press the button down slightly with your finger so as to enable phonation by means of the closure ☀.

**CAUTION!**

Do not use any fatty creams!

Otherwise, secure attachment is no longer ensured.

**V. SERVICE LIFE**

The maximum period of use / wearing period is 24 hours.

The tracheostoma protection can also be changed several times a day, as necessary.

Deviations from the expected wearing period occur as a result of individual wearing conditions and skin properties.

**VI. LEGAL NOTICES**

The manufacturer Andreas Fahl Medizintechnik-Vertrieb GmbH accepts no liability for damage caused by unauthorised product modifications, repairs not carried out by the manufacturer or by unsproper handling, care (cleaning/disinfection) and/or storage of the products contrary to the provisions of these instructions for use.

Should a serious adverse event occur in connection with this product of Andreas Fahl Medizintechnik-Vertrieb GmbH, this is to be reported to the manufacturer and the competent authority of the Member State to which the user and/or patient are domiciled.

The manufacturer reserves the right to make changes to the product at any time without notice.

LARYNGOFIX®/TRACHEOPIX®/TRACHEOSTOMA-FIX®/LARYTAPE® is a trademark and brand of Andreas Fahl Medizintechnik-Vertrieb GmbH, Cologne registered in Germany and the EU member states.

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© 2014 Andreas Fahl Medizintechnik-Vertrieb GmbH
Proizvajalec Andreas Fahl Medizintechnik-Vertrieb GmbH neruči za škodi zplošbeno svetovnimi