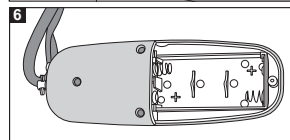
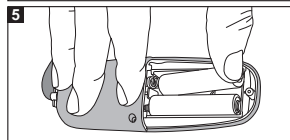
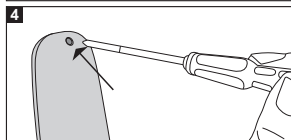
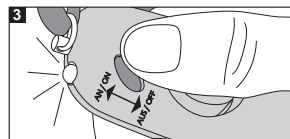
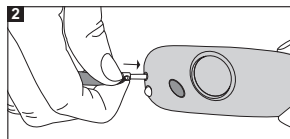
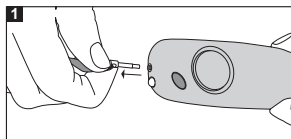
LARYVOX® ALARM

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Manufacturer

Keep dry

Please observe the laws and regulations which apply to the disposal of electrical and electronic devices in the respective countries.

LARYVOX® ALARM

I. FOREWORD

These instructions for use are valid for the LARYVOX® ALARM (REF: 76000).

The instructions for use serve to inform the physician, nursing staff and patient/user in order to ensure technically correct handling.

Please read the instructions for use carefully before using the product for the first time!

II. INTENDED USE

The LARYVOX® ALARM operates as alarm and as flashlight. It enables laryngectomees to draw attention to themselves by an acoustic and optical signal respectively in emergency.

III. PRECAUTIONS

ATTENTION!

The LARYVOX® ALARM is equipped with a high frequency-siren of 98 decibel capacity.

To avoid hearing defects the LARYVOX® ALARM should be used far as possible away from the ear.

IV. INSTRUCTIONS

Alarm:

For activating the alarm pul/ out the plug fastened above the device with the cord ❶. Alarm rings out automatically and without break until the plug is placed in its former position again ❷.

Flashlight:

The device can be used as flashlight by keeping pressed the lateral located switch. By releasing the switch the light is turned off again ③.

Batteries:

The device is equipped with two AAA micro-batteries (1,5 V). For replacing the batteries, the screw / located at the back of the LARYVOX® Alarm has to be removed ④. The new batteries then can be inserted in the battery-pocket ⑤. Pay attention to the correct polarity of the batteries. For closing the battery-pocket the screw is screwed up tightly again ⑥.

V. CLEANING AND DISINFECTION**1. Cleaning**

Wipe the housing of the LARYVOX® ALARM carefully with a moist cloth.

A mild, pH-neutral washing lotion can be used for cleaning.

2. Disinfection

Disinfection can be performed by wiping, using commercial alcohol-based surface disinfectants (for example Sagrotan). Use a soft, lint-free cloth for cleaning.

VI. LEGAL NOTICES

The manufacturer Andreas Fahl Medizintechnik-Vertrieb GmbH will not accept any liability for damages (especially not for functional deficiencies, injuries, infections, and/or other complications or adverse events) caused by unauthorised product alterations, by repairs not performed by the manufacturer or by improper handling, care (cleaning/disinfection) and/or storage of the products in violation of the provisions of these instructions for use. This applies – to the extent permitted by law – both to damages to the products themselves caused thereby and to any consequential damages caused thereby. Product specifications described herein are subject to change without notice. LARYVOX® is a trademark and brand of Andreas Fahl Medizintechnik-Vertrieb GmbH, Cologne registered in Germany and the EU member states.

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