I. VORWORT

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EN. I. FOREWORD

These instructions for use are valid for tube holders. The instructions for use serve to inform the physician, nursing staff and patient/user in order to ensure technically correct handling. Please read the instructions for use carefully before using the product for the first time!

II. INSTRUCTIONS USE

The tube holder allows the tracheostomy tube to be fixed securely in place and can be adjusted to fit different necksizes.

The suffix PED identifies products for paediatric use.

III. WARNINGS

The tube holder is a single-patient product and thus intended exclusively for use - including repeat use - and fits one patient. The tubeholder should not be fixed too tight or too loose around the neck. There should ideally be a distance of approx. 1 cm between tube holder and neck. To prevent the tracheostomy tube from coming off by accident, the tube holder must be checked regularly to make sure it is still securely fixed in place.

IV. INSTRUCTIONS

Place the tube holder around your neck with the hook-and-loop fasteners facing outward. **A**. Attach the two hook-and-loop fasteners to the lateral eyelets on both sides of the neck flange. **B**. Adjust the tube holder (A) with the stretch insert to the required neck size and fasten it at the fleecy side of the holder (**B**). Note that the holder (**B**) is too long it can be shortened with a pair of scissors to the required length. **C**.

V. PERIOD OF USE

The tube holder is a single-patient product and also suitable for repeated use.

The service life depends on the individual frequency of use as well as the condition and on individual handling of the product.

When changing sizes, the tube holder in use must be placed temporarily on a clean surface.

Before being reattached, the product must be checked for visible damage and for functionality.

The product may not be reused if it is damaged, heavily soiled or heavily soaked, as this could jeopardise safe use.

In this case, the product must be replaced immediately.

The tube holder must be disposed of and replaced with a new tube holder no later than 7 days after first being used.

VI. LEGAL NOTICES

The manufacturer Andreas Fahrl Medizintechnik-Ventrib GmbH accepts no liability for damage caused by unauthorised product modifications, repairs not carried out by the manufacturer or by improper handling of the product. The manufacturer is not responsible for storage of the product contrary to the present instructions for use. Should a service advisor come into contact with this product of Andreas Fahrl Medizintechnik-Ventrib GmbH, this is to be reported to the manufacturer and the competent authority of the Member State in which the user/and/or patient are domiciled. The manufacturer reserves the right to make changes to the product at any time without notice. OPTIFLAUSCH® is a trademark of Andreas Fahrl Medizintechnik-Ventrib GmbH, Cologne, which is registered in Germany and the European Member States.

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