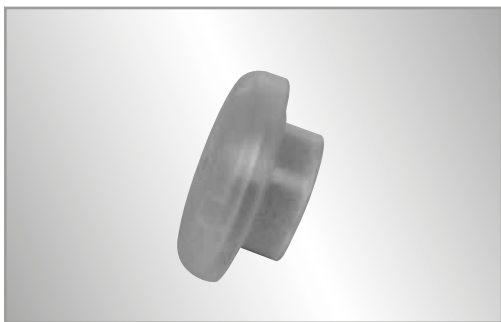
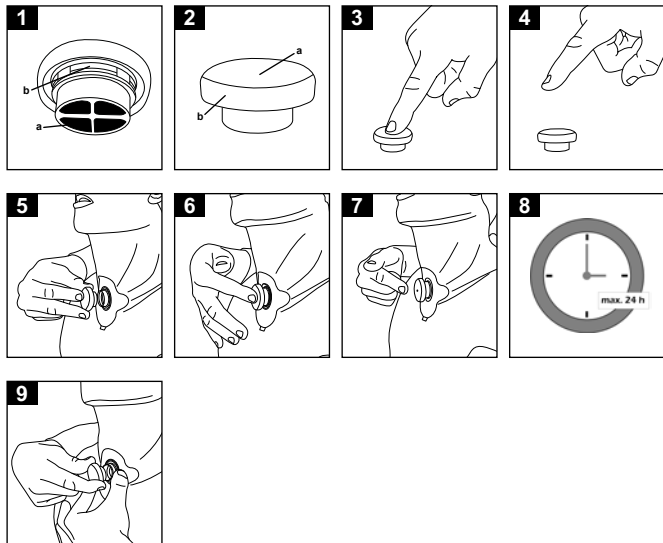
LARYVOX® TOUCH HME

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LARYVOX® TOUCH HME

I. FOREWORD

These instructions for use are valid for LARYVOX® TOUCH HME filter cassettes.

The instructions for use serve to inform the physician, nursing staff and patient/user in order to ensure technically correct handling.

Please read the instructions for use carefully before using the product for the first time!

II. INTENDED USE

LARYVOX® TOUCH HME filter cassettes are particle filters consisting of a foam filter and a plastic housing.

The LARYVOX® TOUCH HME filter cassettes are affixed over the tracheostoma either on a special base plate with holding ring or on a tracheostomy tube with 22-mm combi-adapter.

LARYVOX® TOUCH HME filter cassettes filter the breathing air, thereby reducing intrusion of particles into the patient's airways.

The filter cassettes reduce the formation of viscous secretions in the lungs. They store the moisture and warmth of the exhaled air in the filter medium and emit these to the inhaled air again during inspiration.

When used for the first time the products must only be selected, used and inserted by clinicians or medical professionals trained in their use.

III. PRODUCT DESCRIPTION

LARYVOX® TOUCH HME filter cassettes feature:

- a foam filter for heat and moisture exchange (1a)
- side openings which enable unhindered, easy breathing (1b)
- a speaking button (2a), by means of which the tracheostoma can easily be closed for speaking
- a finger groove (2b) to enable easy removal of the HME filter cassette from the tracheostoma housing for coughing

IV. WARNINGS

The LARYVOX® TOUCH HME filter cassettes are only and exclusively allowed to be used by patients with partially or fully preserved larynx as well as persons wearing voice prostheses (shunt valves)!

LARYVOX® TOUCH HME filter cassettes are single-patient products and are intended for single use only. They must not be cleaned or disinfected, as use of disinfectant can render the filter unusable.

Sterilisation and reuse can impair the function and is therefore not permissible either.

The LARYVOX® TOUCH HME filter cassette must under no circumstances be inserted directly into the tracheostoma!

V. COMPLICATIONS

If the particle filter is clogged or if signs of clogging appear, the LARYVOX® TOUCH HME filter cassette must immediately be removed and disposed of.

VI. CONTRAINDICATIONS

The LARYVOX® TOUCH HME should be removed if respiratory resistance is too high.

Use in patients with strong exsiccosis (dehydration), or excessive secretion in the lungs and airways, or strong air leakage flow (exhaled air not flowing through the filter cassette) is not permissible.

Not for use in patients with impaired consciousness and/or in non-independent persons requiring the help of others, who are not able to remove the LARYVOX® TOUCH HME filter cassette by themselves in emergency situations.

VII. INSTRUCTIONS

1. Check whether the packaging and the product are in a faultless condition.
2. Check the use-by or expiry date. Do not use after this date.

3. Press the speaking button down in order to check whether it is fully functional (⑥). When the speaking button is released, it immediately returns to the open starting position (④).
4. Fit ⑥ the LARYVOX® TOUCH HME filter cassettes vertically onto the 22-mm connection of the tracheostomy tube or base plate, exerting gentle pressure.
5. Check the respiratory resistance.
6. For speaking, press the speaking button ⑥ down slightly with your finger so as to enable phonation by means of the closure.
7. When the speaking button is released, it immediately returns to its open breathing position (starting position) ⑦.
8. Remove the LARYVOX® TOUCH HME filter cassettes after a ⑥ wearing period of 24 hours at most, or if the filter cassette is contaminated with secretions for instance. To do this, hold the base plate/tracheostomy tube and the finger groove in place with two fingers ⑧ and remove the filter cassette.

ATTENTION

If the speaking button (②a) comes off the filter housing (①a) while doing this, then the speaking button can be fixed in place again on the filter housing by applying gentle pressure.

VIII. SERVICE LIFE

The maximum period of use / wearing period is 24 hours.

The filter must be changed even several times a day if required, for instance if the respiratory resistance is in danger of increasing due to accumulation of secretions.

IX. LEGAL NOTICES

The manufacturer Andreas Fahl Medizintechnik-Vertrieb GmbH will not accept any liability for damages (especially not for functional deficiencies, injuries, infections, and/or other complications or adverse events) caused by unauthorised product alterations, by repairs not performed by the manufacturer or by improper handling, care (cleaning/disinfection) and/or storage of the products in violation of the provisions of these instructions for use.

This applies – to the extent permitted by law – both to damages to the products themselves caused thereby and to any consequential damages caused thereby.

Product specifications described herein are subject to change without notice.

LARYVOX® is a trademark and brand of Andreas Fahl Medizintechnik-Vertrieb GmbH, Cologne registered in Germany and the EU member states.

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